

# Instructions for use


## Product description

# Alcohol Surface Disinfectant


Date of revision: 17/04/20 v1.2

topdental™  
Bossklein®


For the surface disinfection of non-invasive medical devices. Specially formulated for use in the Medical and Dental environment.

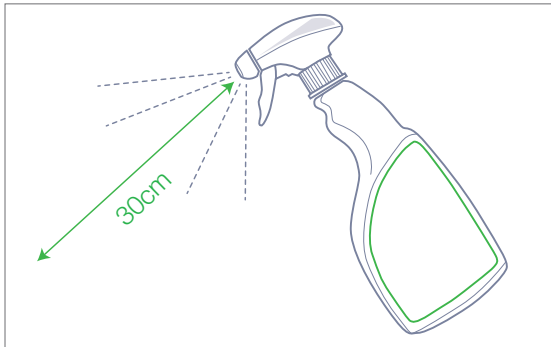

1. Spray surfaces and objects to be disinfected from a distance of approx. 30cm so that they are completely moistened.

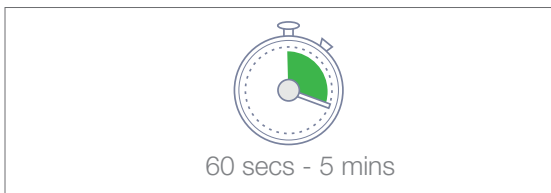

2. Let the disinfectant take action for a minimum of 60 seconds (to achieve full efficacy leave for 5 minutes).

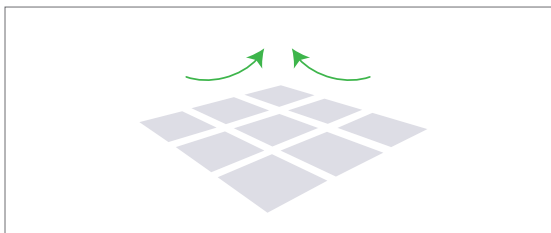

3. Allow surface spray to evaporate.



4. Do not use on surfaces sensitive to alcohol. For professional use only. See product labels and MSDS for further safety guidelines and storage conditions. Available at [topdental.co.uk](http://topdental.co.uk)

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

Composition: Ethanol, Aqua, Propan-2-ol, Polyhexamethylene biguanide hydrochloride, Parfum, Linalool, Limonene.

Danger. EUH208: Contains poly hexamethylene biguanide hydrochloride. May produce an allergic reaction. H225: Highly flammable liquid and vapour. H319: Causes serious eye irritation. H412: Harmful to aquatic life with long lasting effects. P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P280: Wear protective gloves/protective clothing/eye protection/face protection. P303+P361+P353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P370+P378: In case of fire: Use alcohol resistant foam or carbon dioxide to extinguish. P235: Keep cool.

