

Instructions for use

Product description


Daily Aspirator Cleaner and Disinfectant

Date of revision: 17/04/20 v1.3


For the daily cleaning and maintenance of dental aspirator systems.
Can be used on wet line, dry line and semi wet/dry line suction systems.

topdentalTM
Bossklein[®]

5L bottle

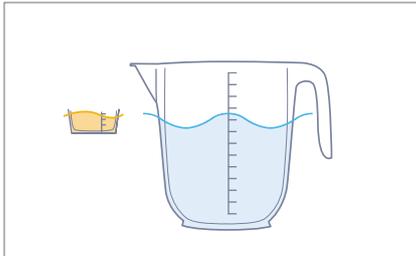

1. Add 20ml ± 1ml of concentrate to 980ml warm water (2% solution).

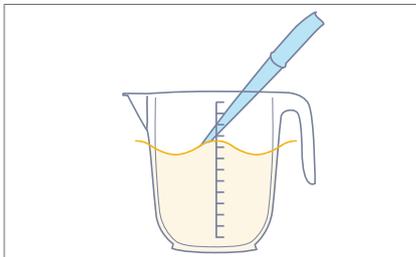


2. Aspirate via the high and low volume suction tubes.




15 - 60 mins

3. Leave for a minimum of 15 minutes (to achieve full efficacy leave for 60 minutes).

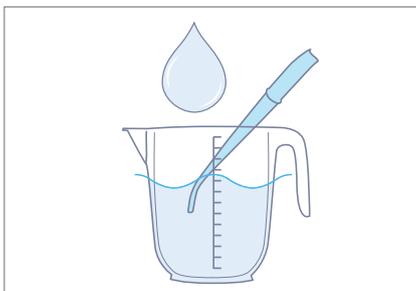


4. Rinse thoroughly with clean water.

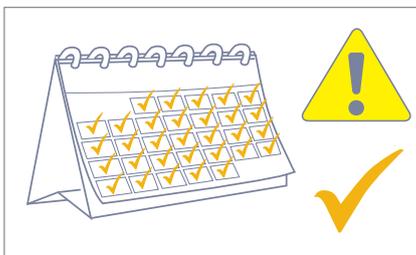


5. Use daily. Alkaline based for removal of Daily deposits. Can be used on wet line, dry line and semi wet/dry line suction systems. Do not use after expiry date. For professional use only. Do not use on aluminium or copper parts. See product labels and MSDS for further safety guidelines and storage conditions. Available at topdental.co.uk

1L dispensing bottle

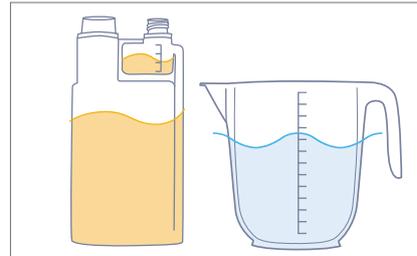

1. Add 20ml ± 1ml of concentrate to 980ml warm water (2% solution) using the dosing device on the bottle.

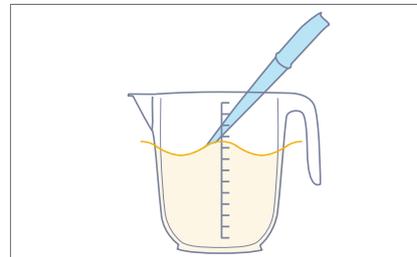


2. Aspirate via the high and low volume suction tubes.




15 - 60 mins

3. Leave for a minimum of 15 minutes (to achieve full efficacy leave for 60 minutes).

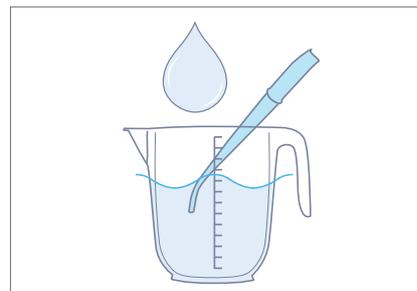

4. Rinse thoroughly with clean water.

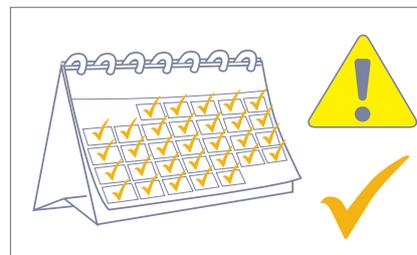

5. Use daily. Alkaline based for removal of Daily deposits. Can be used on wet line, dry line and semi wet/dry line suction systems. Do not use after expiry date. For professional use only. Do not use on aluminium or copper parts. See product labels and MSDS for further safety guidelines and storage conditions. Available at topdental.co.uk

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

Composition: Dimethyldioctylammonium chloride, Dipropylene glycol, 2-Aminoethanol, Tetrasodium ethylene diamine tetraacetate, Ethanol, Citric acid, Nonionic surfactant, Antifoam, Dye.

Danger. (Applies to concentrate, not diluted solution). H314: Causes severe skin burns and eye damage. P260: Do not breathe dust/ fumes/gas/mist/ vapours/spray. P280: Wear protective gloves/protective clothing/eye protection/face protection. P301+330+331: IF SWALLOWED: rinse mouth. Do NOT induce vomiting. P303+361+353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P304+340: IF INHALED: Remove person to fresh air and keep comfortable for breathing. P305+351+338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

 For efficacy details and additional languages see www.topdental.co.uk

 1639