
Alcohol Free Surface Disinfectant

For the surface disinfection of non-invasive medical devices. Specially formulated for use in the Medical & Dental environment. Contains detergent.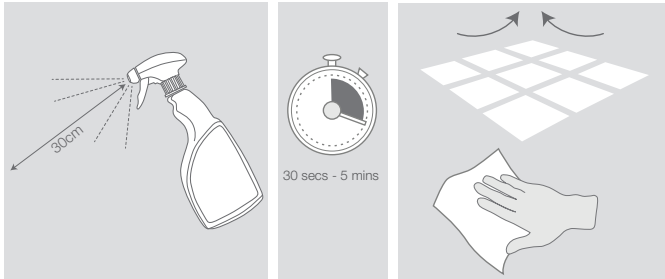

Directions for use: Spray surfaces and objects to be disinfected from a distance of approx. 30cm so that they are completely moistened and let the disinfectant take action over a minimum of 30 seconds (to achieve full efficacy leave for 5 minutes). Allow surface spray to dry or wipe with clean tissue. For professional use only.

Efficacy: Bactericidal (30secs). MRSA, VRE (60 secs). Legionella pneumophila (5 min). Yeastical (30secs). Fungicidal (30secs). Mycobacterium avium (2 min). H1N1 Influenza A (2 min). Human Influenza B virus (30secs). Rotavirus (30secs). HIV (30secs). Hepatitis B Virus (60 secs).

Tested to standards: EN 14885, EN 14476, EN 14348, EN 13727, EN 13704, EN 13624.

Composition: Aqua, Benzyl-C12-18-Alkyldimethyl Chlorides, C12-14-Alkyl[(Ethylphenyl)Methyl]Dimethyl Chlorides, Didecylmethylammonium Chloride, 2-Propanol (not an active, <1%), Surfactant, Dipropylene glycol monomethyl ether, Polyhexamethylene biguanide hydrochloride, Cocamidopropyl Betaine, Tetrasodium EDTA, Antifoam, Preservative, Aroma.

Warning. P101: If medical advice is needed, have product container or label at hand. P102: Keep out of reach of children. P301+P312: IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. The wearing of protective gloves is recommended.

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