Surface Disinfectant

For the surface disinfection of non-invasive medical devices. Specially formulated for use in the Medical & Dental environment.

Directions for use: Spray surfaces and objects to be disinfected from a distance of approx. 30cm so that they are completely moistened and let the disinfectant take action for a minimum of 60 seconds (to achieve full efficacy leave for 5 minutes). Allow surface spray to evaporate. Do not use on surfaces sensitive to alcohol. For professional use only.

Efficacy: Bactericidal (60 secs), including MRSA, VRE. Yeastcidal (60 secs). Mycobactericidal (60 secs). Tuberculocidal (60 secs). Fungicidal (5 mins). Viruses to include (60 secs): Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus (HDV), Human Immunodeficiency Virus (HIV), Influenza Virus (human, avian and porcine), Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles.

Tested to Standards: EN 14885, EN 14476, EN 14348, EN 13727, EN 13624.

Composition: Ethanol, Aqua, Parfum, Linalool, Limonene.

Danger: H225: Highly flammable liquid and vapour. H319: Causes serious eye irritation. H412: Harmful to aquatic life with long lasting effects. P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P280: Wear protective gloves/protective clothing/eye protection/face protection. P303+P361+P353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P370+P378: In case of fire: Use alcohol resistant foam or carbon dioxide to extinguish. P235: Keep cool.

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