
Daily Aspirator Cleaner and Disinfectant

For the daily cleaning and maintenance of dental aspirator systems. Can be used on wet line, dry line and semi wet/dry line suction systems.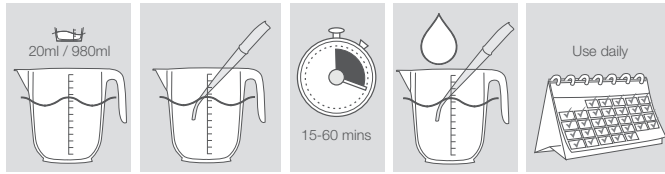

Instructions for use Daily: Add 20ml ± 1ml of concentrate to 980ml warm water (2% solution). Aspirate via the high and low volume suction tubes. Leave for a minimum of 15 minutes (to achieve full efficacy leave for 60 minutes) and then rinse thoroughly with clean water. Do not use on aluminium or copper parts. For professional use only.

Effective Spectrum: Bactericidal (15mins) including Salmonella typhimurium, Legionella pneumophila. Yeastcidal (15mins). Hepatitis B virus (15mins), Hepatitis C virus (15mins), Human Immunodeficiency Virus (15mins). Tuberculocidal (60mins). Fungicidal (30mins).

Tested to Standards: EN14885, EN14476, EN14348, EN13727, EN13697, EN13624, ASTM E1052.

Composition: Dimethyldioctylammonium chloride, Dipropylene glycol, 2-Aminoethanol, Tetrasodium ethylene diamine tetraacetate, Ethanol, Citric acid, Nonionic surfactant, Antifoam, Dye.

Danger. (Applies to concentrate, not diluted solution). H314: Causes severe skin burns and eye damage. P260: Do not breathe dust/ fumes/gas/mist/vapours/spray. P280: Wear protective gloves/protective clothing/eye protection/face protection. P301+330+331: IF SWALLOWED: rinse mouth. Do NOT induce vomiting. P303+361+353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P304+340: IF INHALED: Remove person to fresh air and keep comfortable for breathing. P305+351+338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

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