

Instructions for use

Product description

Alcohol-Free Surface Disinfectant Wipes

Date of revision: 17/04/20 v2.2

topdentalTM
Bossklein[®]

For the surface disinfection of non-invasive medical devices. Specially formulated for use in the medical and dental environment. Contains detergent.


User Tub




1. Remove cap of dispenser tub.




2. Cut open air tight plastic pouch across the top of the pouch.

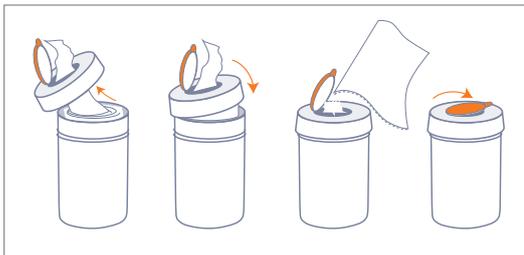

3. Feed the centre wipe through the hole in the cap and replace. Pull wipe at a 45° angle.

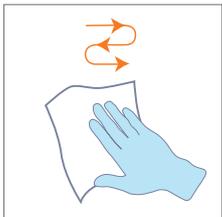


4. Using a new wipe for each application wipe objects to be disinfected so that they are completely moistened.

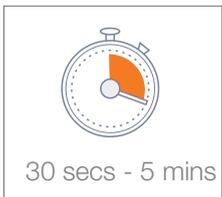

30 secs - 5 mins

5. Let the disinfectant take action for a minimum of 30 seconds (to achieve full efficacy leave for 5 minutes), allowing surface moisture to evaporate, or wipe with clean tissue. For professional use only. See product labels and MSDS for further safety guidelines and storage conditions. Available at topdental.co.uk


Refill Pouch

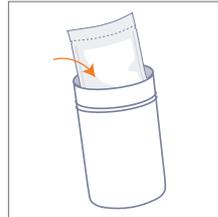


1. Place bag in dispenser tub.




2. Cut open air tight plastic pouch across the top of the pouch.

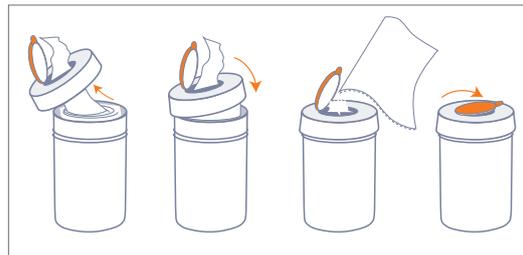

3. Feed the centre wipe through the hole in the cap and replace. Pull wipe at a 45° angle.

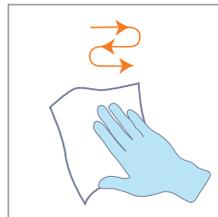

4. Using a new wipe for each application wipe objects to be disinfected so that they are completely moistened.

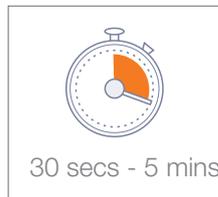

30 secs - 5 mins

5. Let the disinfectant take action for a minimum of 30 seconds (to achieve full efficacy leave for 5 minutes), allowing surface moisture to evaporate, or wipe with clean tissue. For professional use only. See product labels and MSDS for further safety guidelines and storage conditions. Available at topdental.co.uk

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

Composition: Aqua, Benzyl-C12-18-Alkyldimethyl Chlorides, C12-14-Alkyl[(Ethylphenyl)Methyl]Dimethyl Chlorides, Didecyltrimethylammonium Chloride, 2-Propanol (not an active, <1%), Surfactant, Dipropylene glycol monomethyl ether, Polyhexamethylene biguanide hydrochloride, Cocamidopropyl Betaine, Tetrasodium EDTA, Antifoam, Preservative, Aroma.

Warning. P101: If medical advice is needed, have product container or label at hand. P102: Keep out of reach of children. P301+P312: IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. The wearing of protective gloves is recommended.

 For efficacy details and additional languages see www.topdental.co.uk

 1639